

TytoCare | Equipment Operating Instructions

This guide provides step by step instructions on how to operate the TytoCare platform, peripheral use and troubleshooting.


Peripheral Overview




TytoCare | Equipment Operating Instructions

Conducting a Visit

1. From the iPad home screen, select the TytoCare icon to launch the application.

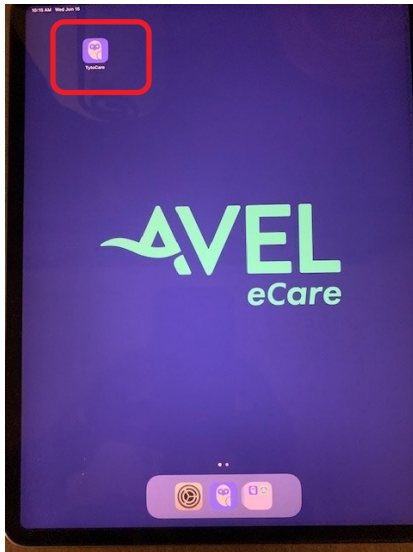


2. Enter Username and Password and select '**Login**'.

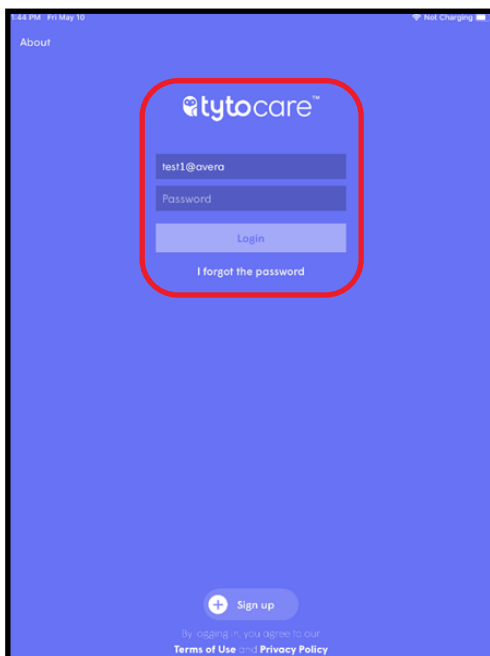


TytoCare | Equipment Operating Instructions

3. In the search field, type the standard patient name assigned to your facility. This name can be found on the **'Login Instructions'** located on the cart.

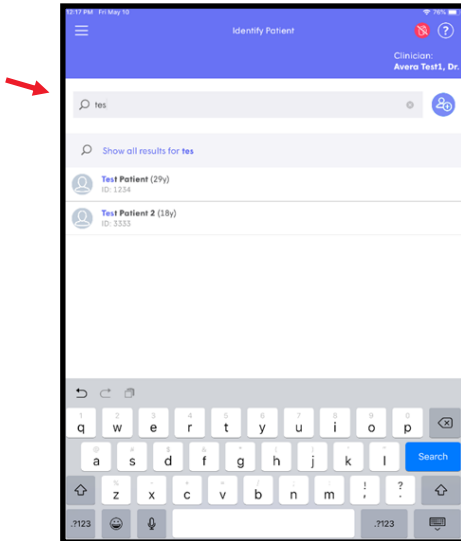


4. Select the accurate name, followed by the **'Select Patient'** icon at the bottom of the screen.

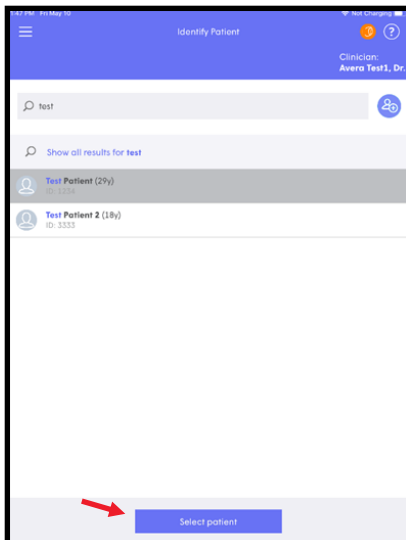


TytoCare | Equipment Operating Instructions

5. Select **'See A Clinician Online'**.

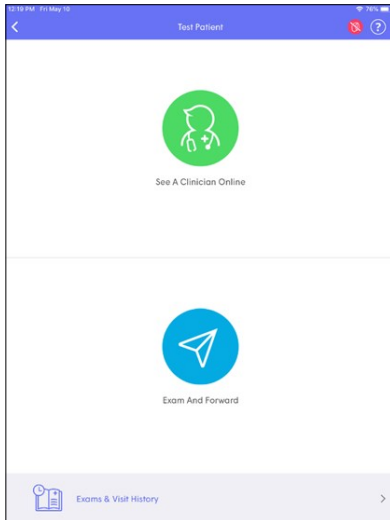


6. Display notification will appear stating **'Medical information is personal. You are about to use the application solution to share it with your clinician.'** Select **'Accept and Continue'**.

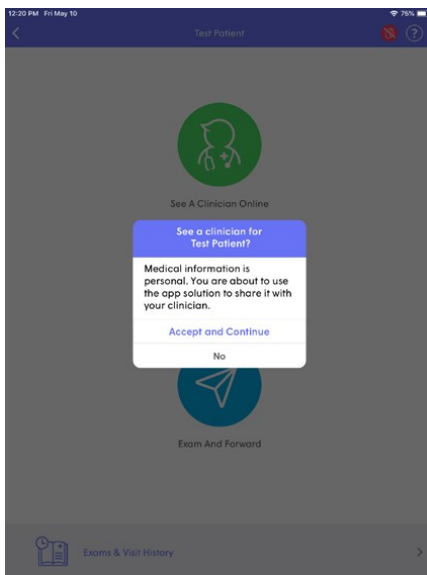


TytoCare | Equipment Operating Instructions

7. Select the provider identified for the patient video encounter.

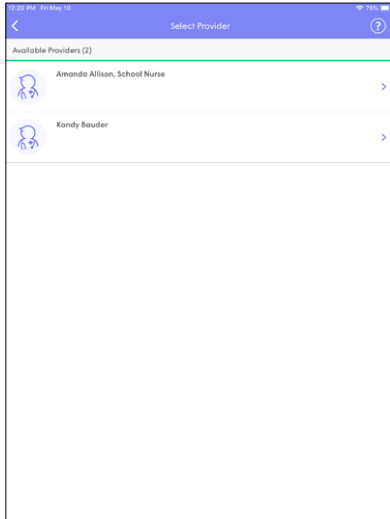


8. This screen will appear while waiting for the provider to acknowledge the video encounter.

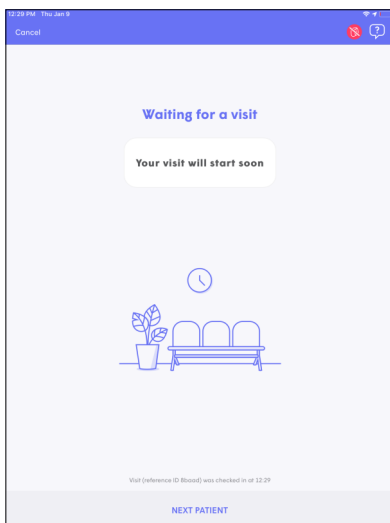

TytoCare | Equipment Operating Instructions

- An audible connectivity alert will be heard when the provider has accepted the request. Select **'Join'** to start the video encounter.

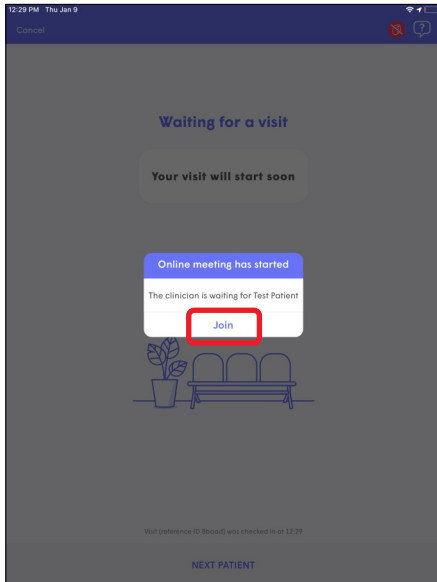

- To confirm the video encounter is active, the provider will appear on the main screen and user/patient will appear in the picture view. This indicates the video encounter is in session.

