

TytoCare | Device User Guide for Exams


Temperature Exams

The infrared temperature sensor is located at the front of the TytoCare device.



1. The patient should be in a relaxed state. Ensure the patients forehead is exposed without interference of hair. Remove excess perspiration from the patients forehead, if needed.



2. When prompted by the provider to begin taking patient temperature, aim the TytoCare device at the center of the forehead, between and above the eyebrows, with a distance of no more than 2 inches (5cm).

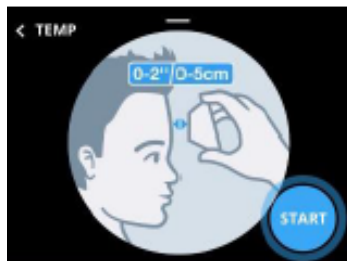

3. The provider will initiate the temperature exam once the TytoCare device is positioned.
4. Hold the TytoCare device bear the forehead during the exam, approximately 6 seconds. An audible beep will indicate completion and the temperature read will appear on the device screen.

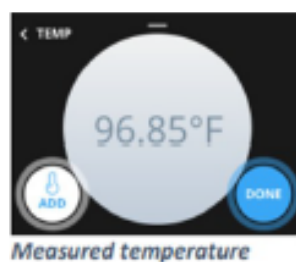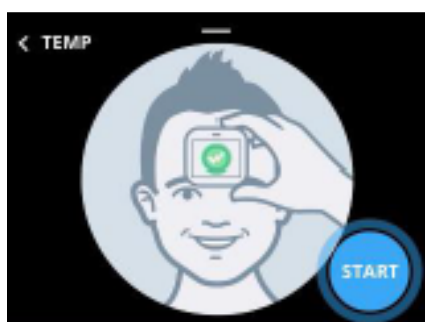


TytoCare | Device User Guide for Exams


Heart and Lung Exams

For the heart and/or lung exam, the prompt will be shown on the device and iPad screen.

1. Connect the stethoscope tip as shown to the device. Align blue dot to blue dot for connectivity use.

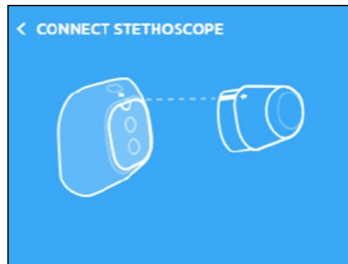

2. Follow verbal instructions from the provider as well as the screen prompts and move the stethoscope to the appropriate area on the patient.

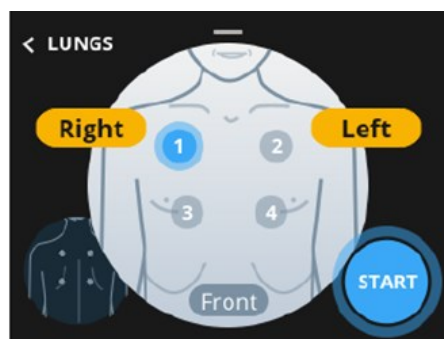

Tyto Display




iPad Display

TytoCare | Device User Guide for Exams


Ear Exams

A prompt on the device screen will be shown for an ear exam.

1. Connect the otoscope tip as shown to the device. Align blue dot to blue dot for connectivity use.

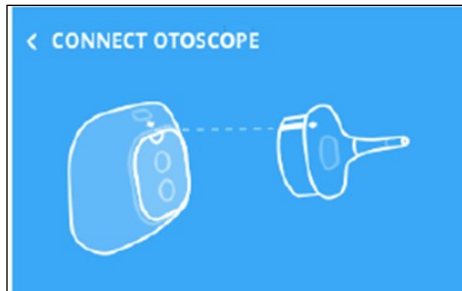

Tyto Device

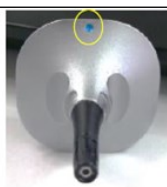


Otoscope Tip



Otoscope Tip and Tyto Device connected

2. Insert the device into the ear canal following the verbal communication from the provider.
3. Maneuver the device to identify best image while watching the video stream on the device.

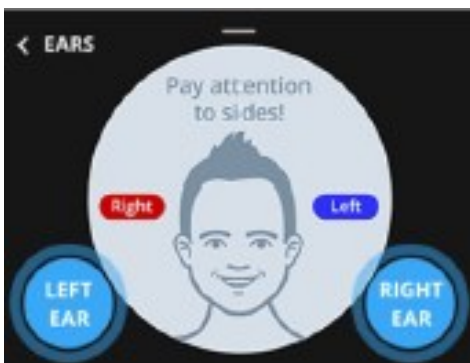

Tyto Display



Example of an eardrum

TytoCare | Device User Guide for Exams

Throat Exams

A prompt on the device screen will be shown for a throat exam.

How to connect the tongue depressor:

1. Attach the holder connector part to the device socket, following the blue dots.





2. Insert the tongue depressor blade into the holder until you hear a locking click.



3. To detach the blade from the holder, press the locking latch at the bottom of the tongue depressor.




4. Instruct patient to open mouth wide while sticking tongue out.
5. Maneuver the device until the back of the throat is clearly visible on the screen.
6. DO NOT throw away holder/connector when disposing of the tongue blade.

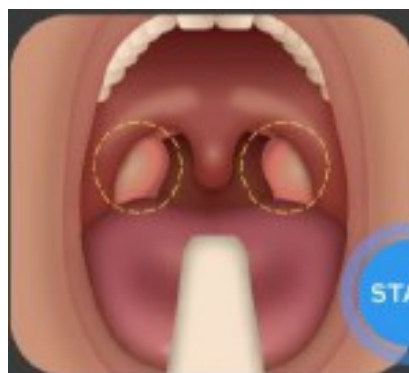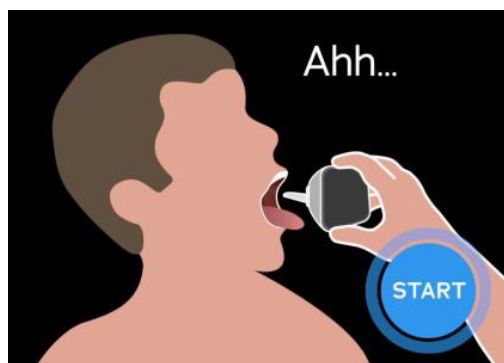

TytoCare | Device User Guide for Exams

Skin Exams

1. Place the TytoCare device approximately 3-5 inches from the skin allowing the camera to focus.

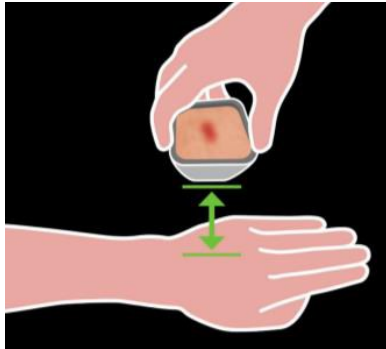

2. To focus the camera, press finger on the device screen to indicate the location the camera should focus.

