

TytoCare | How to End a Video Encounter

Note: If the TytoCare device will not be in use within 4 hours or longer, power down device and plug into power.

1. After the provider ends the video encounter, the image below will be displayed on the iPad screen. Select **'OK'** to end the session.

The of	A	alatad
The ci	the visit.	piered
wifi	OK n test Provi	der

2. To logout of the TytoCare application, select the three lines in the top left corner. Select 'Logout'.

=	Identify Patient	• · · · · • • • • • • • • • • • • • • •
		Clinicion: test overa
O Type patient's name or ID		8
	Select patient	

K 333 AM ² - Mine May 13	Menu	₩ 3394 (¥
User Profile		
Change Passoord		
Pair Tyto Device		
Support		
? User Monuol		
Test Connection		
Choose Longuage		
O Unit Soffings		
,		
	lagnut	



TytoCare | How to End a Video Encounter

3. The window display will prompt **'Are you sure you want to logout?'** Select **'Yes'** and the main login screen will display.

9:30 AM Mon May 13	Menu	🕈 100% (1983)
User Profile		
Change Passw	ord	
Pair Tyto Devic	e	
Support		
? User Manual	Are you sure you want to lagout? No Yes	
Test Connectio	in	
Choose Langue	oge	
Unit Settings		
	Logout	

About		Not Charging I
	%tyto care"	
	test1@avera	
	Login	
	🛨 Sign up	
	By logging in, you agree to our Terms of Use and Privacy Policy	

4. The TytoCare device screen will prompt **'Online visit is finished'**. Select **'Finish'** and the main TytoCare screen will display.





- 5. When the TytoCare platform is not in use for patient care:
 - a. The iPad is required to be powered on
 - b. The TytoCare device is required to be plugged into the power charge cord and turned off
 - c. The cart is required to be plugged into an electrical outlet